

# Specimen Instructions

## FFPE Specimens (Block or Slides)


**Note: Do NOT USE** strong acids (e.g. hydrochloric acid, sulphuric acid, picric acid) as these destroy nucleic acid. When decalcification is required, brief exposure to a weak acid such as EDTA is recommended.

### SAMPLE TYPE

#### 1 FFPE BLOCK OR 16 UNSTAINED SLIDES (+ 1 H&E SLIDE)

Tissue should be formalin-fixed and embedded into a paraffin block. Use standard fixation methods with 10% neutral-buffered formalin.


DO NOT use other fixatives (AZF, B5, Bouin's, Holland's). If sending slides, send 16 unstained slides (charged and unbaked, with tissue cut at a 5 micron thickness), plus 1 H&E slide.

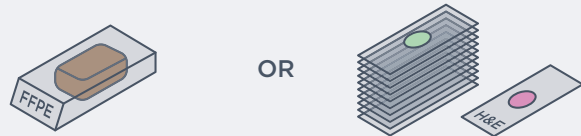

### SURFACE AREA

#### 2 Optimum: 5 x 5 mm<sup>2</sup>

Tissue should have a surface area of at least 25 mm<sup>2</sup> (5 x 5 mm<sup>2</sup>, 2.5 x 10 mm<sup>2</sup>)

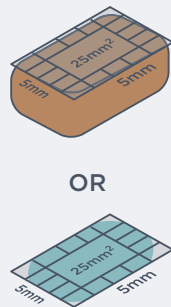


### SURFACE VOLUME

#### 3 Optimum: 2 mm<sup>3</sup>

Optimal sample volume can be achieved by sending optimal tissue surface area (25 mm<sup>2</sup>) at a depth of ≥80 microns.


For suboptimal tissue surface area, additional depth is required.

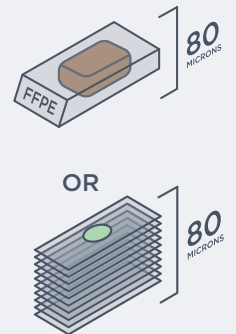

### NUCLEATED CELLULARITY

#### 4

DNA is extracted from nucleated cells. Samples with low nucleated cellularity (e.g., those with abundant mature erythrocytes, lesional cells that contain excessive cytoplasm, or tissue with extensive associated fibrosis) may require greater tissue volume to yield sufficient DNA at extraction.

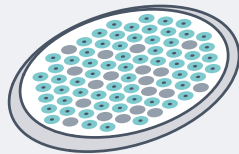


### TUMOUR CONTENT

#### 5 Minimum: ≥20%

If the ratio of nucleated malignant to nucleated non-malignant cells is too low, sensitivity of detection of certain classes of alterations is reduced. High tumour content is preferable.

**Note for liver specimens:**

Higher tumour content may be required because hepatocyte nuclei have twice the DNA content of other somatic nuclei.

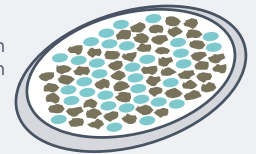

**Note:** All cytologic and histologic specimens will be reviewed internally by a pathologist and a determination of sample adequacy will be made. Additional or alternate material may be requested for optimal analysis.

## Shipping instructions

1. Place the samples, FoundationOne®Heme requisition form and any other attachments into the FoundationOne Heme FFPE Specimen Shipping Kit.
2. Place the specimen shipping kit (including samples and paperwork) into the provided courier shipping pack, first ensuring that primary specimen containers (e.g. blocks, slides) are labeled with two patient-specific identifiers. Seal the shipping pack.
3. Complete the pre-printed shipping labels (if necessary) and apply to shipping pack.
4. Call the local courier to request a pick-up and sealed shipping pack to:

***Foundation Medicine, Inc.***  
***7010 Kit Creek Road***  
***Morrisville, NC 27560***  
***Phone: +1.888.988.3639***